Modelo: iWear Supersoft
REF Troquel: TRO-PROS-014
Dimensiones: 540 x 175 mm

Código: PROS589.2

Colores:


black

PROS589.2

iWear®

175 mm

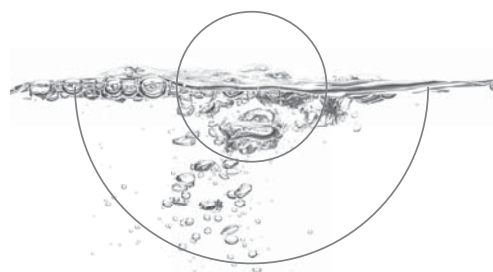supersoft

all-in-1

gb

Multipurpose Solution with Aloe Vera for silicone hydrogel lens users. Cleaning and deposit elimination (proteins and lipids), disinfection (effective against bacteria, fungi and Acanthamoeba), rinsing, storing, wetting, lubricating.

INSTRUCTIONS FOR USE:

- Before handling your lenses, wash your hands with soap and water.
 - Place your lens on the palm of your hand and rub it carefully with a few drops of **iWear® Supersoft** solution, on both sides for a few seconds.
 - Rinse both sides of the lens for at least 5 seconds using **iWear® Supersoft** solution.
 - Place your lenses in the lens case, and be sure to cover these with **iWear® Supersoft** solution. Close the case and shake it gently for a few seconds.
 - Keep lenses at least 6 hours in **iWear® Supersoft** solution (or overnight) to grant an adequate disinfection.
 - Once the disinfection process is done, you may insert your lenses directly to your eye or you may rinse them previously with **iWear® Supersoft** solution.
 - Rinse the lens case with **iWear® Supersoft** solution and allow it to air dry.
- Last revision date: 2012/01

PRECAUTIONS:

- Once opened, the bottle should not be used after 90 days.
- Maintain the bottle well closed after use.
- Do not use the product if the safety seal is missing or broken.
- You may store your lenses up to 30 days in the closed lens case.
- Do not ingest the solution and keep away from children's reach.
- Keep at room temperature.
- Do not allow the dropper to come into contact with any surface to avoid contamination.
- In case of irritation, consult your contact lens specialist.
- Before using any ophthalmic medicine, inform your doctor you are a contact lens user.
- Do not use if you are allergic to any of the components.
- Never use tap water to rinse your lenses or lens case.
- Do not use after the expiration date.
- Discard the solution in the lens case after each use.
- Your specialist will be able to recommend additional products or changes in cleaning procedures in line with your tear evaluation and the kind of lenses you use and their replacement frequency. Always follow your professional's instructions and never change them without first consulting him/her.

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