easydrops  
eyedrops

**gb**

**Usage:**  
iWear® Easydrops soothe the symptoms of ocular irritations, dryness and tiredness. Its special formula contains Sodium Hyaluronate, a natural substance present in the eye, that forms a uniform and protective shield with a great water absorption capacity and resistant to evaporation. The iWear® Easydrops provide your eyes with an excellent hydration thus improving visual comfort and lens wear. They are especially recommended for contact lens users in environments where ocular irritations and dryness increase.

**Instructions for Use:**

- Wash your hands with soap and water before using the product.
- Direct use: Apply one or two drops into each eye and blink several times.
- Indirect use: Apply one or two drops into the internal part of each lens before putting them on.
- Use iWear® Easydrops as many times as needed.

Last revision date: 2009/11

**Precautions:**

- Do not use the product if the safety seal is missing or broken.
- Do not let any surface come into contact with the nozzle to prevent contamination.
- Do not use the product after it has been opened for over 60 days, nor after the expiry date.
- Keep well closed after each use.
- Do not ingest the solution and keep out of children's reach.
- Before using any ophtalmic medicine, inform your doctor you are a contact lens user.
- Do not use the product in case you are allergic to any of its components.
- In case of ocular irritation or adverse reaction please consult a specialist.
- Store at room temperature.

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AVIZOR S.A.